

Erasing Recordings

To Erase a Level—

1. Press and release the REC button. The Play/Record LED will be solid red.
2. Find the level you want to erase via the Volume up button ▲.
3. Press the Play button twice within one second to erase the level message.
4. Press and release the REC button to exit Record mode.

To Erase a Step within an Activity Sequence—

1. Locate the desired activity (A,B,C) via the 1,2,3 button.
2. Press and release the REC button. The Play/Record LED will be solid red.
3. Locate the step you want to erase via the Volume up button ▲.
4. Press the Play button twice within one second to erase the step.
5. Press and release the REC button to exit Record mode.

To Erase All Steps of an Activity

1. Press and release the REC button. The Play/Record LED will be solid red.
2. Locate the activity sequence you want to erase via the 1,2,3 button.
3. Press and hold the REC button for five seconds. A beep lets you know when the activity sequence is erased.
4. Press and release the REC button to exit Record mode.

Level/Activity Lock

Lock the Big Button on the current Level and Activity by disabling the 1,2,3 button. To disable the 1,2,3 button, turn the device off, press and hold the Volume down button ▼, and turn the device on. Release the Volume down button ▼ when two ascending beeps play, indicating the Level and Activity Locks are on. To unlock, repeat the directions above. Two descending beeps indicate Level and Activity Locks are off.

Record Lock

To disable Record, turn the device off, press and hold the Volume up button ▲, turn the device on. Release the Volume up button ▲ when two ascending beeps play, indicating the Record Lock is on. To unlock, repeat the directions above. Two descending beeps indicate Record Lock is off.

Quick Start

Basics

- On/Off switch on the back
- 2 AAA batteries included
- LED for Steps and Levels emits through the transparent cap
- REC button on the back
- Adjustable volume
- Threaded insert and keyhole on back for mounting
- Overlay storage on back

Operations

- Choose between Steps and Levels with a switch on the back
- Select from 10 levels with the 1,2,3 button on the back
- Select from 3 activities (A,B,C), also with the 1,2,3 button
- Record up to 10 steps per activity
- Press and hold the Play button continuously for 5 seconds to reset the Steps operation.
- Edit any message without affecting other messages

Touch the edge or top to play a message



Switch accessible



Attainment Company's

Big Button

with Steps and Levels

Ten levels—each with 8 seconds recording time.
Three activities—each with ten 8-second steps.

Speaker

Play button

easy-touch access!

Holds 3" x 3" images

Weight: 8.1 oz. (.23 kg)

Size: 5 1/8" across (14.6 cm)

Switch jack

Volume buttons

Microphone

Record/Play LED

Steps and Levels LED

User's Guide



Attainment Company, Inc.

P.O. Box 930160 • Verona, WI 53593-0160 • USA

Phone 1-800-327-4269 • International 1-608-845-7880

www.AttainmentCompany.com

The **Big Button with Steps and Levels** is a powerful voice output tool with a single large Play button. A removable transparent cap snaps onto the Play button and has an opening on top for sliding in overlays or icons. Remove the cap by firmly pulling up on the lip until it dislodges. Then pull on the top of the cap to remove it. The cap extends nearly to the table surface, creating a large activation area with an extremely low profile. In fact, you can play a message without lifting your hand off the table. Only a little pressure is needed to activate speech.

Low-profile,
easy-touch
activation!



Volume

Volume up and down buttons are on the front right side. The Big Button beeps when you adjust the volume up or down.



Switch accessible!
The switch jack is located
on the Button's right edge.
When connected, your switch
performs the same functions
as the Play button.

Levels and Steps Operations

Choose from two operational styles—**Levels** or **Steps**—with the toggle switch on the back. The Steps and Levels LED shows through the transparent cap.

In **Levels** operation, the same message plays each time the Play button is pressed. Select a level with the 1,2,3 button on the back.



In **Steps** operation, a different step of the activity sequence plays each time the Play button is pressed. The device has a ten-step capacity for each of three activities. Press and hold the Play button continuously for 5 seconds to reset the device to Step 1.

Removable,
transparent cap for
easy cleaning



Large,
easy-touch
Play button

Levels Operation Recording


1. On the back of the device, move the Step/Level switch to **Level**.
2. Press and release the REC (Record) button. A solid red LED on the front indicates your Big Button is ready to record.
3. Press the Volume up button  to locate the level you want to record. The Level LED will show 1–9 and then L in sequence. The L stands for last.
4. Press and release the Play button and the red LED will flash. You are now recording. Speak clearly, about 8" from the Big Button. *Note: The click you hear when you press the Play button will not record.*
5. Press and release the Play button when you are done recording. Otherwise, recording will automatically stop after 8 seconds. The LED will return to solid red.
6. Press and release the Volume down button  to hear your recording. To re-record, follow recording directions 3–5.
7. To exit Record mode, press and release the REC button. The red LED will turn off.

Note: You can erase or re-record any level message without affecting other messages.

Playing


Press and release the Play button. The Record/Play LED will flash green during playback. Press and release the Play button during playback to stop the message. Only one message will play until you change levels via the 1,2,3 button.

Steps Operation Recording

1. Move the Step/Level switch to **Step**.
2. Press and release the 1,2,3 button to select one of the three activities (A,B,C). You can record up to ten steps for each activity.
3. Press and release the REC button. A solid red LED will turn on, and "1" is shown in the Steps LED. You are now ready to record Step 1.
4. Press and release the Play button and the red LED will flash. You are now recording Step 1. Speak clearly, about 8" from the Big Button. *Note: The click you hear when you press the play button will not record.*
5. Press and release the Play button when you are done recording. Otherwise, recording will automatically stop after 8 seconds. The LED will turn to solid red and the Step LED will advance to 2.
6. Press and release the Volume down button  to hear what you have just recorded. To re-record Step 1, cycle through the steps by pressing the Volume up button. Stop on "1" and record.
7. Follow recording directions 4–5 to record additional steps.
8. To exit Record mode, press and release the REC button. The red LED will turn off.

Note: You can erase or re-record any step without affecting other messages.

Playing

Locate the desired activity sequence (A,B,C) via the 1,2,3 button. To play the first step, press and release the Play button. The Record/Play LED will flash yellow. If the step is played to completion, you automatically advance to the next step. To hear the previous step again, press and release the Volume down button  immediately. To stop the playback of a step, press and release the Play button. You do not advance to the next step if playback is interrupted. The Steps LED will display an L with a corresponding beep to indicate you are at the end of an activity sequence. Press and hold the Play button continuously for 5 seconds to return to Step 1.

Press and hold the Play button continuously for 5 seconds to reset the Steps operation.

