



**Attainment
Company, Inc.**

GoTalks Attainment Company's Cleaning and Disinfecting Instructions

Spray cleaning solution onto a soft non-abrasive cloth; do not spray cleaning solution directly onto a GoTalk. Use the dampened soft cloth to wipe and clean the GoTalk.

Attainment Company suggests non-alcohol based glass cleaner or 1:3 vinegar to water solution. Although, any commercially available disinfectant or a diluted bleach solution is fine to use as long as the cleaning liquid stays on the outside of the device.

- Do Not submerge GoTalks in water/liquid
- Do Not clean GoTalks in dishwashers
- Do Not open GoTalks for cleaning or any other purpose
- Do Not spray cleansers directly onto GoTalks
- Do Not use abrasive cleaners

If a GoTalk is submerged in water or an excessive amount of liquid is spilled on the device, immediately open the battery compartment and remove the batteries. We also suggest hanging the device by the handle or standing the device up so liquid will not pool inside the device.

Attainment is happy to provide the support you need. Contact Attainment Company's technical support team with your questions or comments.

Email: techsupport@AttainmentCompany.com

Toll-free phone: 800-327-4269 ext. 4510

Outside North America phone: +1-608-845-7880