

Attainment's

GoTalk[®] WOW

for the Apple[®] Watch

USER GUIDE



LINKED TABLE OF CONTENTS

Introduction

Apple Watch Information

Pair Apple Watch to iPhone

Buttons and Force Press

Quick Start

Edit/Add/Delete

Edit a Talking Point

Add a Talking Point

Delete a Talking Point

Settings

Back Up

Restore from Backup

Tips for Using Images

Send Images from iPhone to Watch

Add Multiple Emojis

Apple Watch Screenshots

Delete Images from Watch

Easy Access to GoTalk WOW

Via Settings

Customize Extra-Large Watch Face

Pair Bluetooth Speaker

Technical Support

Introduction

Welcome to **GoTalk WOW**, an easy to use AAC app designed specifically for the Apple Watch.

Enjoy quick access to customized messages, called *Talking Points*, right on your wrist. Just tap to play!

You can play your Talking Points on the Apple Watch, even when your iPhone is in another location.

A basic set of Talking Points are included. And you can use your own custom images or emojis to create new ones on the go!



Apple Watch Information

Pair Apple Watch to Your Phone

To use **GoTalk WOW**, your Apple Watch must be set up and paired to your iPhone.

Turn on the Watch and hold it close to your iPhone, and step-by-step instructions will show up on the iPhone.

For detailed instructions:

<https://support.apple.com/en-us/HT204505>

Buttons and Force Press


GoTalk WOW uses the Apple Watch's digital crown and Force Press gesture frequently. The Side Button also has useful functions.

- **Digital Crown:** Name for the scroll wheel on the side of the Watch. In **GoTalk WOW**, roll to scroll up or down through Talking Points. Press to function as "Back" or "Home" button.
- **Side Button.** Press to view icons of apps available on the Watch.
- **Force Press** gesture. The Apple Watch is pressure-sensitive. To **Force Press**, hold your finger on the watch screen and press down firmly until new window appears.



Quick Start

- On launch of **GoTalk WOW**, a Talking Point will display.
- Roll the **Digital Crown** up or down to scroll through the Talking Points.
- Tap the screen to activate a Talking Point.
- Playback will stop when watch screen goes to sleep.
- **Force press** on any Talking Point to edit/add/delete messages or access Settings.

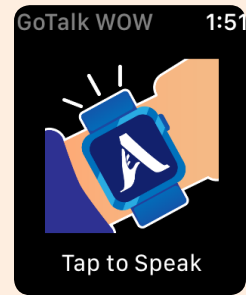


Edit/Add/Delete

Talking Point messages have audio that plays when tapped, and visible image, text, or both.

The **GoTalk WOW** menu allows you to edit existing Talking Points, add or delete one, or change settings for the app.


To access this menu, **Force Press** on a Talking Point.

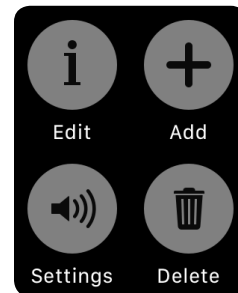

Edit A Talking Point

Force Press on an existing Talking Point to view the menu.

- Tap on **i/Edit** to make changes to the audio and visual message. See **On Tap**, **Visual**, **Text**, and **Cue** below.
- Scroll down to the bottom of the menu and tap **Save** to save any/all edits.

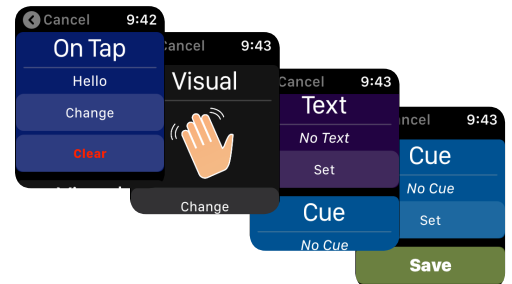

On Tap – Change Talking Point Audio

NEW TEXT-TO-SPEECH MESSAGE

- Tap **Change**
To *dictate* your message, tap the microphone icon. Speak into the watch and see your words translated into text. Tap **Done** when finished. *Remember, the change will not be saved until you scroll to the bottom and tap **Save**.*
- To *write* your message, tap the hand icon. Use your finger to draw a single letter. When you remove your finger, the letter will appear at the top of the screen. Tap **Space** to add a space between words. Tap **Done** when finished. To save edit, scroll down and tap **Save**.
- Tap **Change** to rewrite your message.
- Tap **Clear** to erase your message.



NEW VOICE RECORDING AUDIO

- Tap **Clear**
Tap **Record Audio**, bring Watch to your mouth, and speak. Press red square to end recording.
- **Clear** to re-record, or **Save**. Scroll down and tap **Save** to save new recording.

NOTE: While you can have unlimited Talking Points, it's best if you can limit your message to 20 seconds or less.

Visual – Change Image or Emoji

Tap **Change** to choose a different image or Emoji

- Use the **Digital Crown** to scroll through available images. Stop on the image you want and then tap **<Visual** at top left to select it. To save new image, scroll down to bottom and tap **Save**.
- To transfer pictures to your Watch, see [Tips for Using Images](#) below.

To access Emojis:

- Use the **Digital Crown** to scroll to the bottom of the image list. Tap on **Set Emoji**.¹
- Tap the **Smiley Face** to access the Emoji sets.² (Previously used Emojis will also display.)
- First set is frequently used Emojis, followed by categories of Emojis.³
- Tap on an Emoji to choose it, then tap **<Visual** at top left.

Text – add text to view on your Talking Point


NOTE: Text is not required. A Talking Point can have an image, text, or both.

- Tap **Change** to replace text, or **Set** to add new text. The method is the same as adding Text-To-Speech. Choose microphone to dictate, or hand to write. Tap **Done** when finished. To save new text, scroll down and tap **Save**.



Cue (optional) – Add audio to play automatically

A Cue will play as soon as you scroll to the Talking Point, no tap required. Cues are abbreviated text-to-speech messages to cue the AAC user, similar to auditory cues in GoTalk NOW.

- Tap **Change** to replace a Cue, or **Set** to make a new Cue. Choose microphone to dictate, or hand to write. Tap **Done** when finished.

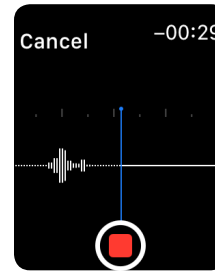


TAP TO
SELECT

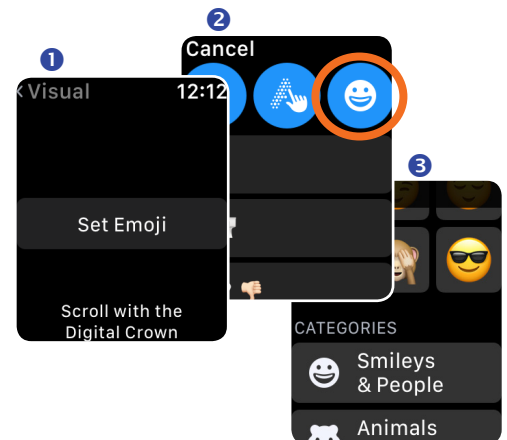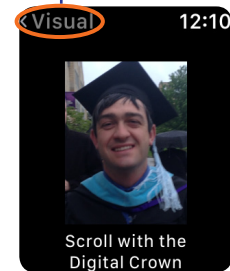

DICTATE
MESSAGE

WRITE
MESSAGE

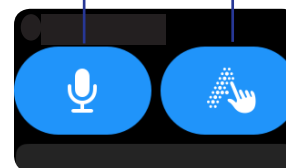

Save – Save the edited Talking Point

In order to save the Talking Point edits, scroll down to the bottom of the edit menu and tap the **Save** button to save your changes.



Add A Talking Point

Force Press on any Talking Point to view the menu.

- Tap **+/Add**.
- Set **On Tap Audio**, **Image**, **Text**, and/or **Cue**.
- Scroll down to the bottom of the menu and tap the **Add** button to save your new Talking Point.



Delete A Talking Point

Force Press on an existing Talking Point and tap **Delete**.



Settings

To access: **Force Press** on any Talking Point and tap **speaker/Settings**.

- **Volume** (system/watch wide): Tap and then use the Digital Crown to adjust
- **Scroll Resistance**: Adjusts the amount the Digital Crown needs to be turned to advance through your Talking Points.
- **Scroll Debounce**: Prevents scrolling to another Talking Point for up to 5 seconds after arriving at a Talking Point.
- **Tap Debounce**: Prevents activating an additional Talking Point for up to 5 seconds after activation.
- **Text to Speech Rate**: Adjusts the speed of the text-to-speech voice.
- **Vertical Swipe Navigation**: Off by default. When turned on, the user can use their finger to swipe up and down through their Talking Points.
- **Longform Audio**: Prevents audio playback from stopping when the Watch screen goes to sleep.


NOTE: Requires a Bluetooth Speaker. See instructions below on pairing a speaker to the Watch.

Back Up

GoTalk WOW will automatically back up every 5 days when your paired iPhone is nearby. Use the backup method below if you make a change and want to immediately save it.

The **Backup** menu pictured at right is a sub-menu of Settings. **Force Press** on any Talking Point and tap **Settings**. Inside the Settings menu, **Force Press** again.


- Tap **Backup** to back up messages, user images, and settings to your iPhone. This option will only appear when paired iPhone is nearby.
- Tap **Delete All** to reset to the default. All new Talking Points will be removed, but user images and built-in Talking Points will still be available. You will need to confirm.

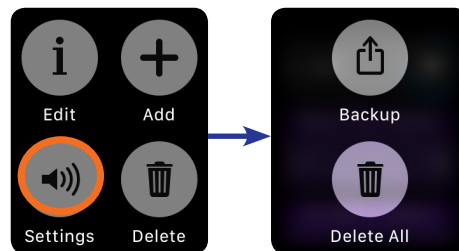

Restore from Backup

1. ON THE IPHONE: Open the **GoTalk WOW** app.
2. Tap the Transfer icon in the top-right corner.
NOTE: The icon only appears when the Watch is active, nearby, and running GoTalk WOW.
3. Tap **Send Restore File**
4. Tap Browse folder (bottom right of phone screen)
5. Tap "On My iPhone" under Locations
6. Tap the **GoTalk WOW** folder. It is automatically generated when you do a backup. Select Restore file. When transfer is complete, a checkmark will appear.
7. ON THE WATCH: **Force Press** on any Talking Point; tap **Settings**; **Force Press** again to access Backup menu.
8. Tap **Restore**. This action will overwrite all current Talking Points and user images with those on the file.


NOTE: The Restore file stays on the Watch and can be used again. The Watch stores one Restore file at a time. A new Restore file sent from the iPhone will replace the first one.

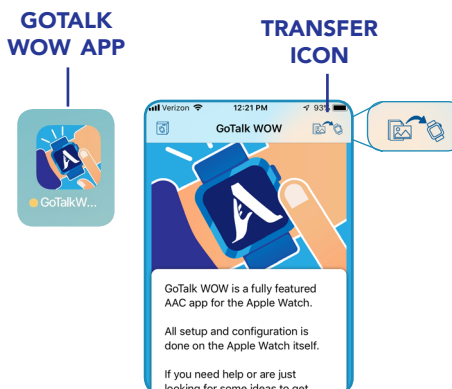

Tips for Using Images


Send Images from iPhone to Watch

1. Open the **GoTalk WOW** app on your iPhone
2. Tap the Transfer icon
NOTE: the Watch has to be nearby the iPhone, awake, and have **GoTalk WOW** running for the Transfer icon to appear on the phone.
3. Tap "Send Photo" and browse images that are saved in the iPhone's Photos app. Tap a photo to send it to the Watch.
4. You can also save images from an Internet website onto your phone, then send them to the Watch.

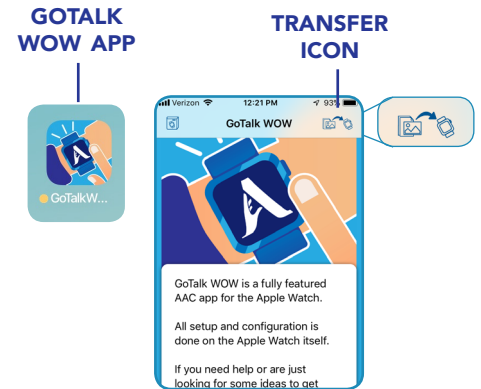

Add multiple Emojis to your Talking Point

Tap **Force Press** on the Change Emoji screen to reveal an "Append Emoji" button. This allows you to have additional Emojis in one message. Find the Emoji(s), then tap **<Visual** to select.

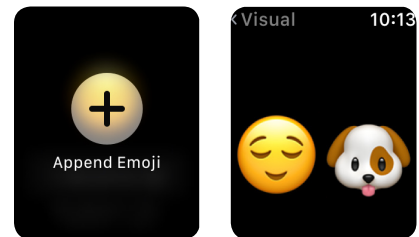

Apple Watch Screenshots

- Screenshots are disabled by default. To enable: Open the Apple Watch app on your iPhone. Under "MyWatch," scroll down to General > Enable Screenshots. Toggle switch on.
- To take a screenshot, press the **Digital Crown** and the **Side Button** at the same time. The screenshot will be saved in both the iPhone's and Watch's Photos.



Delete Images from Watch

Scroll to the image and **Force Press** on it, then tap **Delete**.


Easy Access to GoTalk WOW

Here are two methods to launch **GoTalk WOW** quickly on your Watch.

Via Settings

- Press **Digital Crown** to see available Apps as icons.
- Tap **Settings** to open
- Tap General > Wake Screen > ON SCREEN RAISE SHOW LAST APP > Always

With this setting, whenever the Watch is active, **GoTalk WOW** will automatically be open.

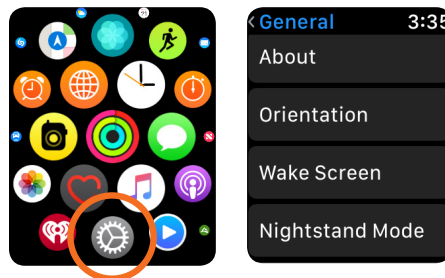

Customize Extra-Large Watch Face

- On Watch, swipe right through watch faces to find extra-large watch.
- **Force Press** on watch face
- Press **Customize**. The first screen is for color.
- Change color, if desired, by scrolling with **Digital Crown**.
- Swipe to the next screen and tap on large rectangle
- Scroll with **Digital Crown** to find the **GoTalk WOW** app, and tap on it
- The revised watch face will show the time and a **GoTalk WOW** icon. Tap on the **GoTalk WOW** icon to open the app.



Pair Bluetooth Speaker

In noisy settings, you may want to boost the volume of your Talking Points by pairing the Watch to a Bluetooth speaker.

- Turn Power on Bluetooth speaker
- Force Press on any Talking Point, tap on Settings
- Tap on Bluetooth. Watch will search.
- Tap "Pairing" button on speaker

Technical Support

Tech Support is available weekdays from 9 am–5 pm (CST)

Email: techsupport@AttainmentCompany.com

Phone: U.S and Canada - (800) 327-4269
International - +1 (608) 845-7880

We welcome your questions and comments:
info@AttainmentCompany.com



www.AttainmentCompany.com

